


PLEASE CAREFULLY READ THE INSTRUCTIONS BEFORE YOU START.
Use these instructions in conjunction with your antigen test kit or PCR test instructions.

RHINOMED

RHINOSWAB JUNIOR SAMPLE COLLECTION INSTRUCTIONS FOR USE

TO BE USED UNDER THE GUIDANCE OF AN ADULT.

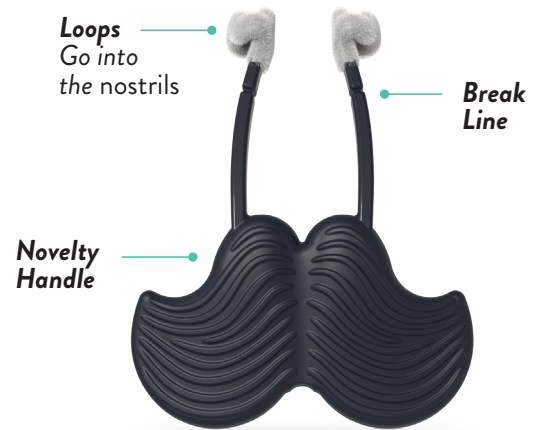


RHINOSWAB
JUNIOR
by RHINOMED®

1: WASH OR SANITIZE YOUR HANDS.




2: TOUCHING ONLY THE HANDLE, remove the swab out of its packaging.

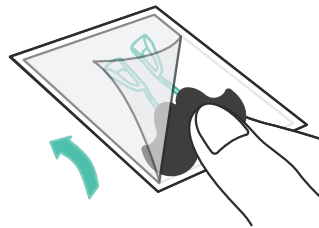


Do not use if pouch is damaged


3: ASK THE CHILD to hold the swab by the handle and bring it to their nose.

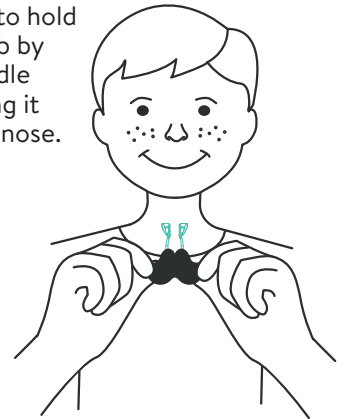

4: THE CHILD SHOULD PLACE THE SWAB gently in their nose as far as is comfortable.

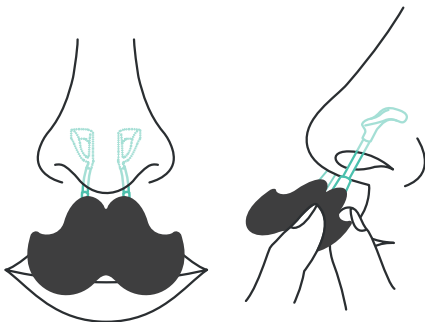

5: SET TIMER FOR 30 SECONDS and ask the child to breathe normally through their nose.

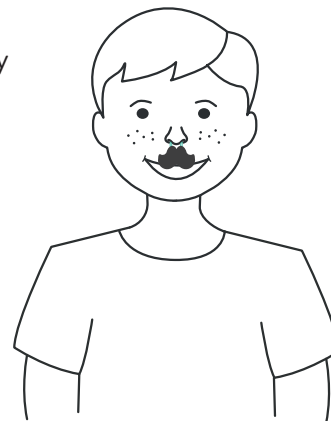

Set timer for 30 SECS

PLEASE TURN OVER


7: AFTER 15 SECONDS, ask the child to move the swab inside their nose for **15 seconds** making sure the loops make continual contact with the inside of their nose.

Move swab inside the nose
15 secs




Remove from the nose.

8: BREAK BOTH LOOPS INTO THE TUBE ONE AT A TIME. Insert one loop at a time into the tube and snap off at the break line.

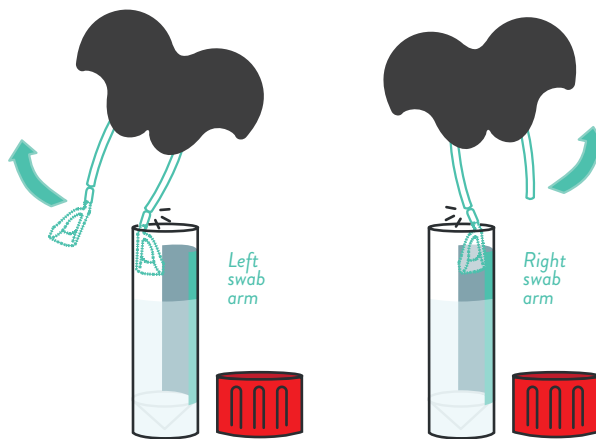

Do this by bending the swab arm against the edge of the tube at the break line.

9: SCREW THE CAP BACK TIGHTLY ON THE TUBE ensuring it is tightly sealed so the liquid does not leak.

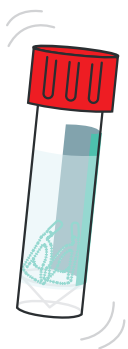

SHAKE THE TUBE GENTLY A FEW TIMES

10: WASH OR SANITIZE YOUR HANDS.



11: PROCESS TEST.

Please follow the instructions to return the sample to the relevant testing center.

Or refer to the instructions on the antigen test kit.

SYMBOL	MEANING
	RHINOMED Level 1, 132 Gwynne St, Cremorne VIC 3121 swab@rhinoswab.global
 Patent Pending	Medical device
	Notified body
	Do not reuse
	Exposure to sunlight
	Use before

SYMBOL	MEANING
	Consult the instructions for use
	Do not use if packaging is damaged
	Unique device identifier
	Individual lot number
	Manufacturer catalogue/reference number
	European authorized representative